


macrofill ^{XL}

vacuum

- Breast reconstruction
- Breast augmentation
- Gluteal augmentation

Volume
150-2000 mL

Process faster very large volumes of fat while respecting the crucial steps of the Adip'sculpt protocol to ensure the high quality of adipose tissue graft

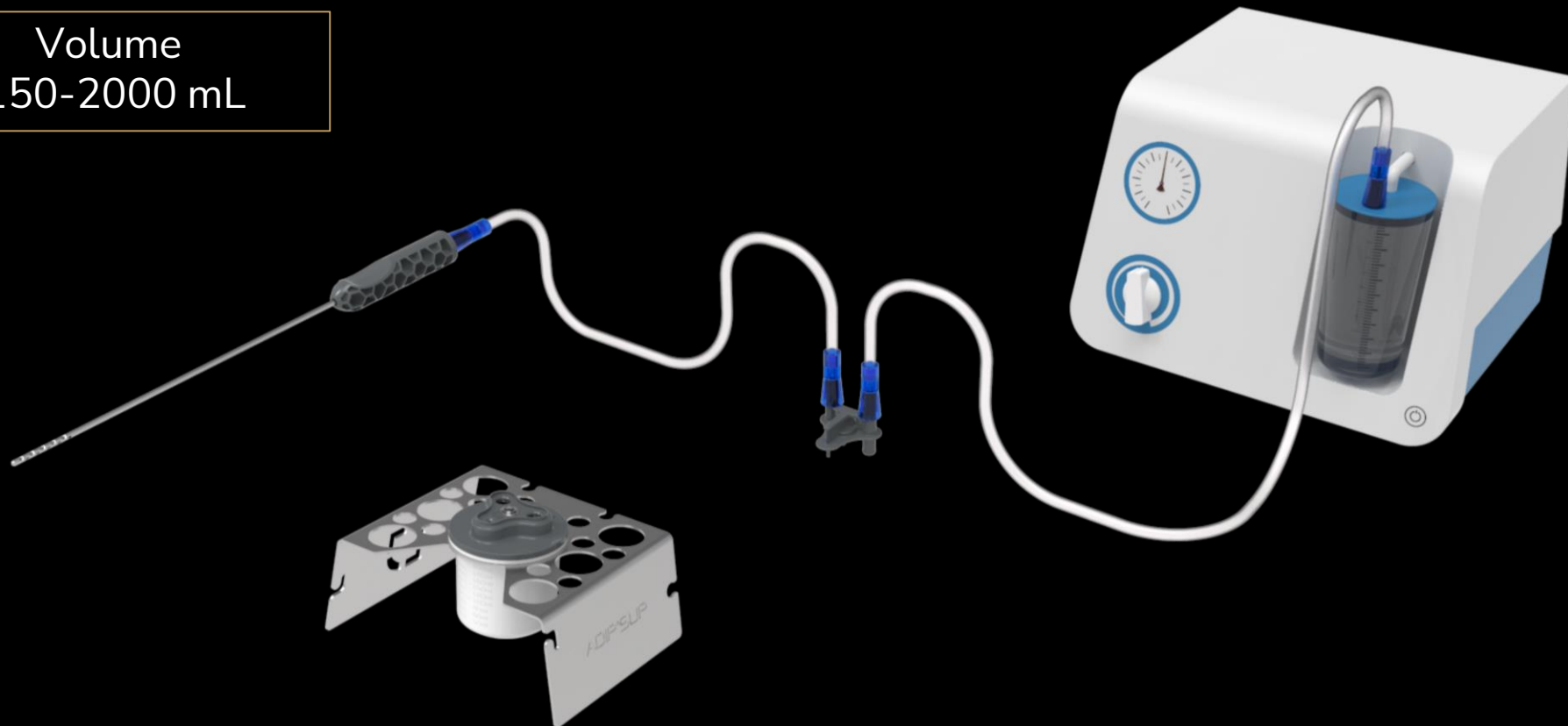

Kit content



① 1 harvesting cannula

② 2 jars with connectors

③ 1 infiltration cannula

④ 1 injection cannula Ø1,7x2,1 L200

⑤ 1 injection cannula Ø1,7x2,1 L130


⑥ 1 transfer cannula

⑦ ⑧ 2 needles (18G L40 & 18G L40 NOKKOR)


⑨ 2 FLL-FLL connectors

⑩ 1 rotor nut

Surgical procedure

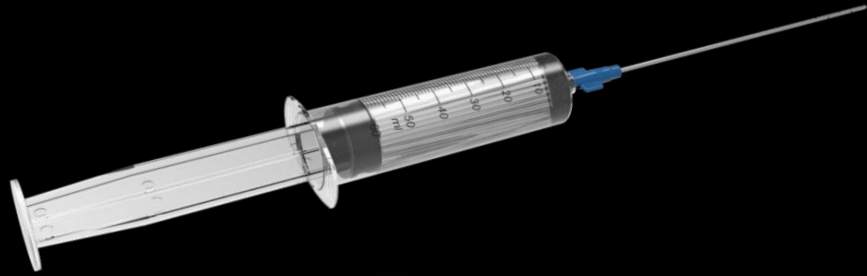

1. Infiltration: The donor site is infiltrated using the infiltration cannula

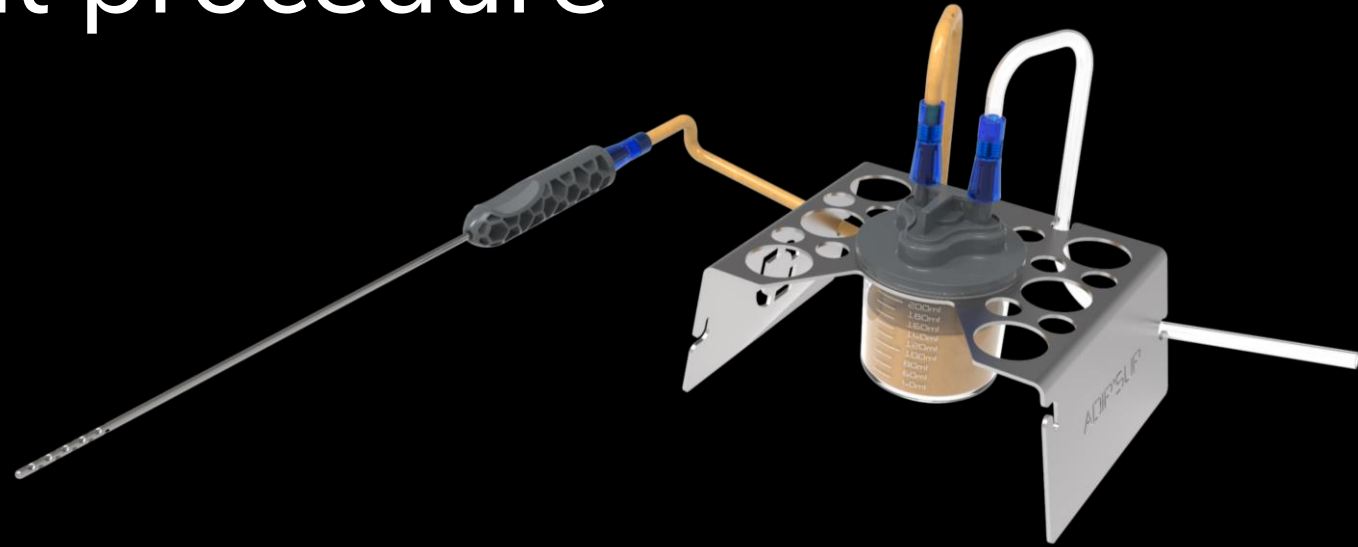

2. Harvesting: The assisted lipoaspiration is performed using a liposuction equipment and the Macrofill vacuum cannula



3. Washing and centrifugation: Macrofill vacuum jars are centrifuged according to Adip'sculpt protocol using the dedicated rotor



4. Injection: The adipose tissue is transferred into 10 mL syringes and injected using the injection cannula

Advantages & Innovation

- **High quality adipose tissue:** All steps of the Adip'sculpt protocol are respected
- **Fast procedure:** Macrofill vacuum will allow faster harvesting and processing of adipose tissue thanks to the use of the liposuction vacuum, enabling surgeons to save time
- **Compatible with conventional liposuction devices:** Surgeons will not need to buy a new one
- **Compatible with the Adip'spin centrifuge:** A specific rotor has been designed for the use of this product
- **User-friendly:** Macrofill vacuum's harvesting cannula is ergonomic and the protocol is easy to implement in the surgeon practice (less steps)
- **Patient safety:** Macrofill vacuum remains microbiologically safe, from the infiltration to the reinjection step
- **Competitive price:** Less expensive than Macrofill® or other devices

Macrofill vacuum range

macrofill
vacuum cannula



Macrofill vacuum cannula
Ref: KTAVACUUM-C10

The harvesting cannula can be bought separately by box of 10 units

macrofill
vacuum



Macrofill vacuum
Ref: KTAVACUUM

All-in-one surgical kit

macrofill
vacuum jar



Macrofill vacuum jar
Ref: KTAVACUUM-J02

The jar can be bought separately by box of 2 units

Contact



Julien GAUTHIER

Managing Director

+ 33 6 80 00 47 34

julien.gauthier@adipsculpt.com

Evgenia TSOY

Export Sales Manager

+ 33 6 58 27 00 32

evgenia.tsoy@adipsculpt.com

www.adipsculpt.com

